



# AP *medical*

## Nursy *line*



Organizzazione con Sistema di Gestione certificato Company with Management System certified

ISO 9001:2000



Disposable and unpiercable containers with hermetic closure for sharp and cutting waste.



# Nursy line

Wide opening with indents for needle's removal

Temporary closure

Definitive safety lock




**NURSY 3 LT**

**NURSY 5 LT**

**NURSY 10 LT**




- Very stabile shape
- Hermetic closure in overturn position
- Easy temporary closure
- Universal trolley fixing system for circular and square tubes
- During incineration no toxic or harmful gas emission occurs
- Ergonomic handle
- Lid hanging device
- Filling level indicated

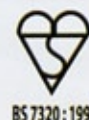

Item	Lt	Max load	Product cm	Weight kg	Pcs/carton	Carton cm	Pcs/pall	Pallet cm
NURSY 3 LT	3	1,35	15,5X15,5X22	0,28	40	79X40X31	720	80X120X220
NURSY 5 LT	5	2,52	22X22X21	0,43	40	79X53X48	320	80X120X210
NURSY 10 LT	10	5,47	22X22X35	0,60	30	79X53X48	240	80X120X210

# Instructions of use

## 1 Lid assembly

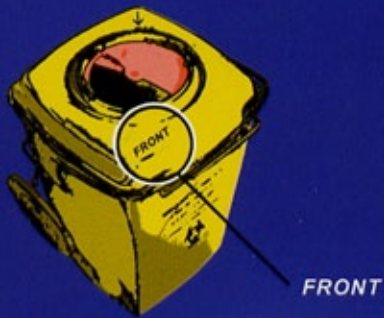

- Assembly the lid (front side) with the body (label side)



- Press firmly on the opposite angles



- Press firmly on the opposite sides

## 2 Needle removal



- Respect the maximum filling level indicated on the container

## 3 Lid closure



- Temporary closure



- Definitive safety lock (arrow towards arrow)

## Supports



- The same support is used to fix the container on the trolleys, horizontally or vertically, by square and/or circular section



- Horizontal fixing



- Vertical fixing

 **AP** medical

Retailer's stamp

